



MIS Impactor



Instructions for use

Positioning and locking the implant

① PREPARATION OF THE IMPACTOR FOR THE FIXING OF THE IMPLANT ⇒ Surgical assistant


Take out the ratchet mechanism

- Press the button (A).
- Pull out the ratchet mechanism (B) completely.

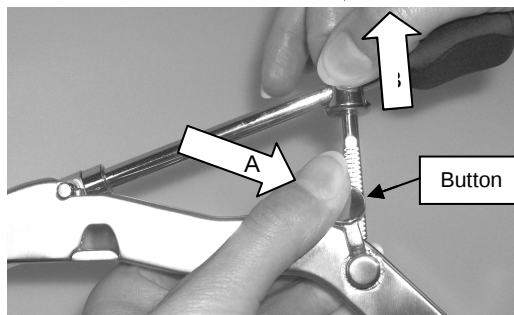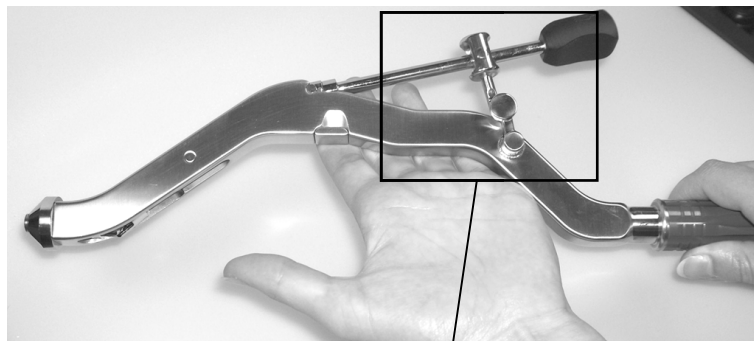



② THE IMPACTOR IS READY FOR THE FIXATION OF THE IMPLANT ⇒ Surgical assistant

The ratchet must be released (C).

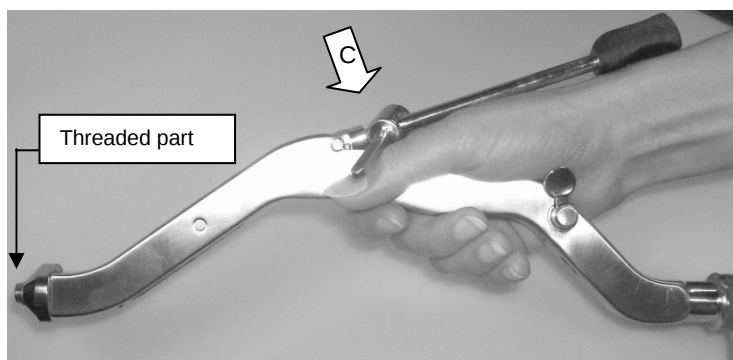

③ FIXATION OF THE IMPLANT ON THE IMPACTOR ⇒ Surgical assistant

- position the threaded part of the impactor in front of the hole in the acetabular shell.
- Screw the impactor drive chain into the implant using the small handle (D).
DO NOT OVERTIGHTEN.

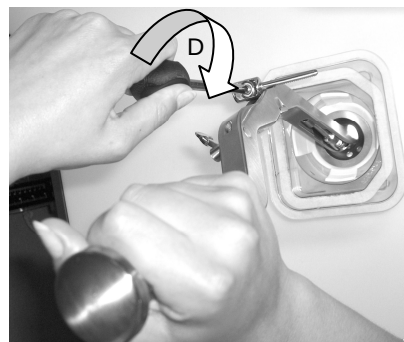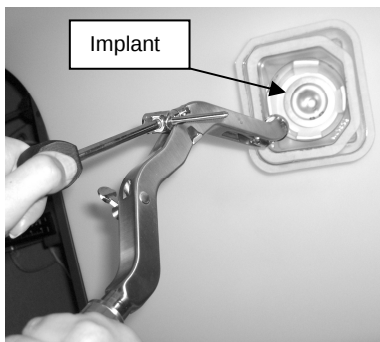




④ INTRODUCTION OF THE RATCHET

⇒ Surgical assistant

- Introduce the ratchet into the matching hole.

IMPORTANT


Attention, count 1-3 clicks after the engagement of the ratchet. The implant must be free to rotate when the handle is turned (see point 5).

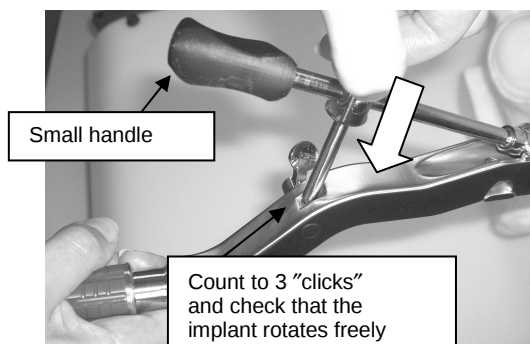

⑤ ORIENTATION OF THE IMPLANT

⇒ Surgeon


- orientate the fixation holes with the help of the rotating knob (E). Turn clockwise so as not to unscrew the implant.

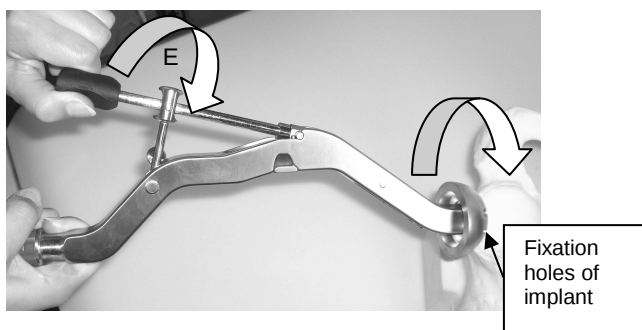

⑥ LOCKING OF THE IMPLANT

⇒ Surgeon

Press down (firmly) on the lever to lock the implant.

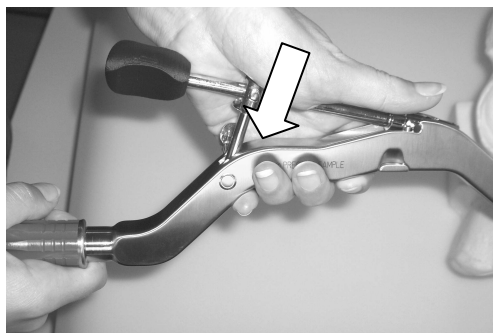

⑦ INSERTION OF THE IMPLANT INTO THE PATIENT

⇒ Surgeon





Greatbatch[™]
Medical



GREATBATCH MEDICAL

10000 Wehrle Drive
Clarence, NY 14031
U.S.A.

Tel. #. +1 716-759-5600

orthopaedicsna@greatbatchmedical.com

EC REP

GREATBATCH MEDICAL SA

7, L'Echelette

2534 Orvin

Switzerland

Tel. # +41 (0)32-358-01-11

orthopaedicseurope@greatbatchmedical.com



D 3.3.1.16




Release of the implant after impaction

①

UNLOCKING THE IMPLANT

After the impaction of the implant :



- press the button of the lever (F),
the ratchet releases
automatically (G).

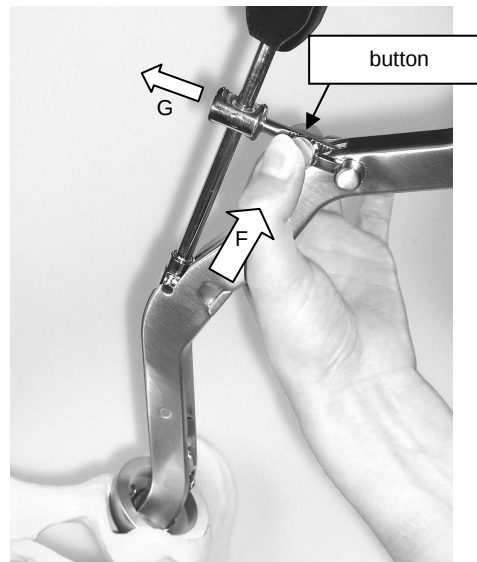

②

RELEASE THE IMPLANT

Unscrew with the help of the small handle (H) the threaded part of the impactor in a counter-clockwise direction. Pull the impactor out.

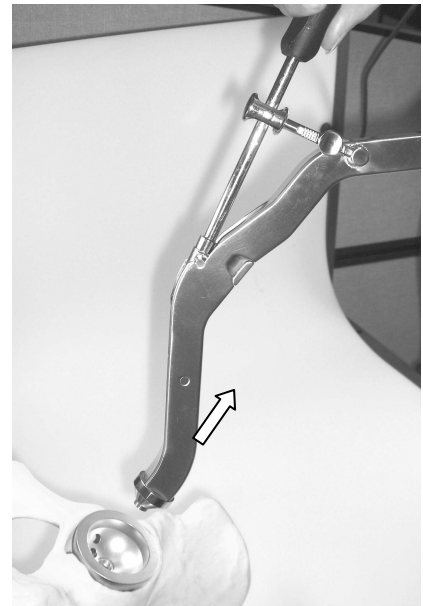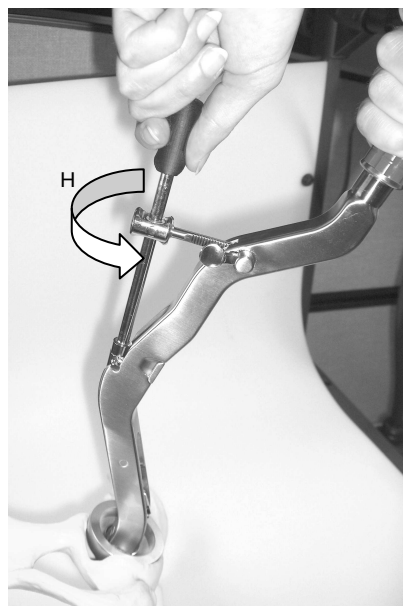



Greatbatch[™]
Medical



GREATBATCH MEDICAL

10000 Wehrle Drive
Clarence, NY 14031
U.S.A.

Tel. #. +1 716-759-5600

orthopaedicsna@greatbatchmedical.com

EC REP

GREATBATCH MEDICAL SA

7, L'Echelette

2534 Orvin

Switzerland

Tel. # +41 (0)32-358-01-11

orthopaedicseurope@greatbatchmedical.com



D 3.3.1.16




Impacteur MIS Cleaning instructions

Disassembly, cleaning and sterilization

①

Disassembly of the ratchet


Press the button (A) and pull out the ratchet (B).

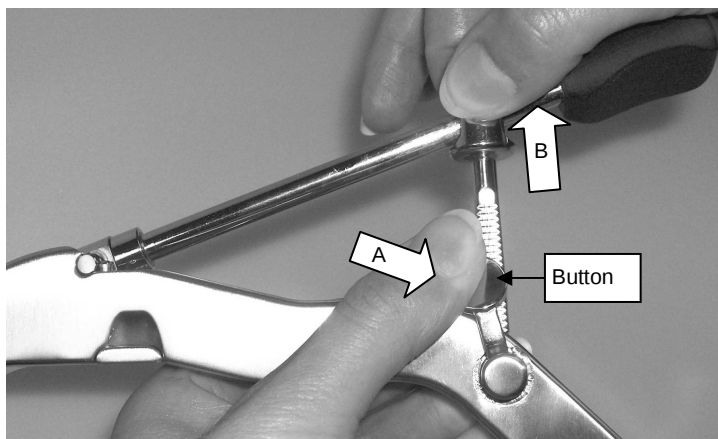

②

Disassembly of the drive chain

Pull down the locking plate (C) and release the drive chain (D).

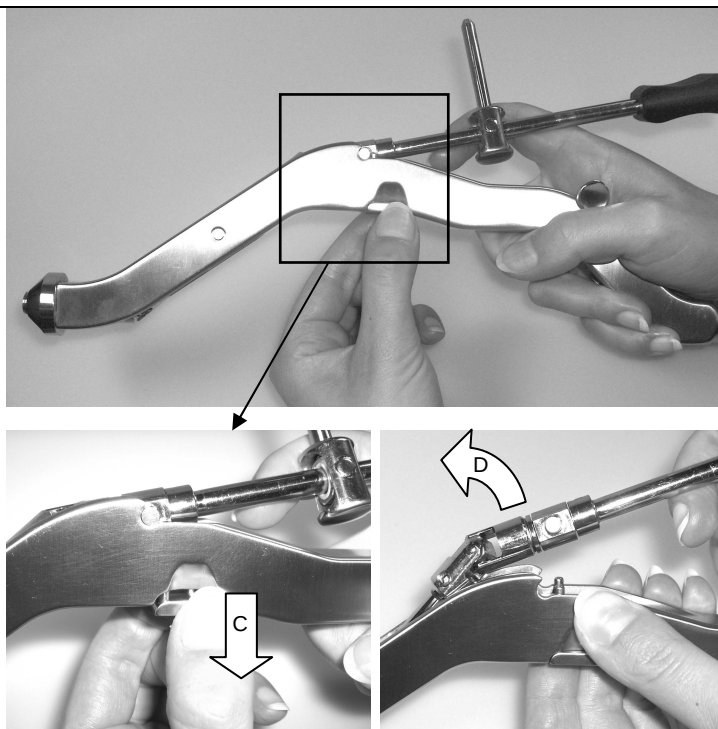



Greatbatch[™]
Medical



GREATBATCH MEDICAL

10000 Wehrle Drive
Clarence, NY 14031
U.S.A.

Tel. #. +1 716-759-5600

orthopaedicsna@greatbatchmedical.com

EC REP

GREATBATCH MEDICAL SA

7, L'Echelette

2534 Orvin

Switzerland

Tel. # +41 (0)32-358-01-11

orthopaedicseurope@greatbatchmedical.com




D 3.3.1.16



③


Take the threaded part out of the body of the impactor

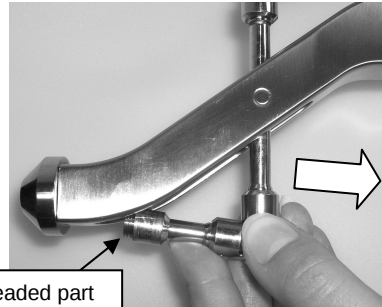

Threaded part

④

Clean and sterilize in this position

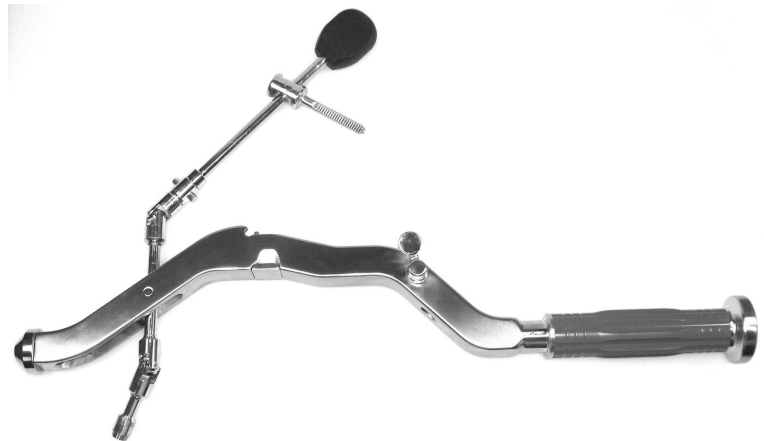



Greatbatch[™]
Medical



GREATBATCH MEDICAL

10000 Wehrle Drive
Clarence, NY 14031
U.S.A.

Tel. #. +1 716-759-5600

orthopaedicsna@greatbatchmedical.com

EC REP

GREATBATCH MEDICAL SA

7, L'Echelette

2534 Orvin

Switzerland

Tel. # +41 (0)32-358-01-11

orthopaedicseurope@greatbatchmedical.com




D 3.3.1.16



Assembly of the instrument after sterilization

① INTRODUCTION OF THE THREADED PART

Slide the threaded part into the body of the impactor.

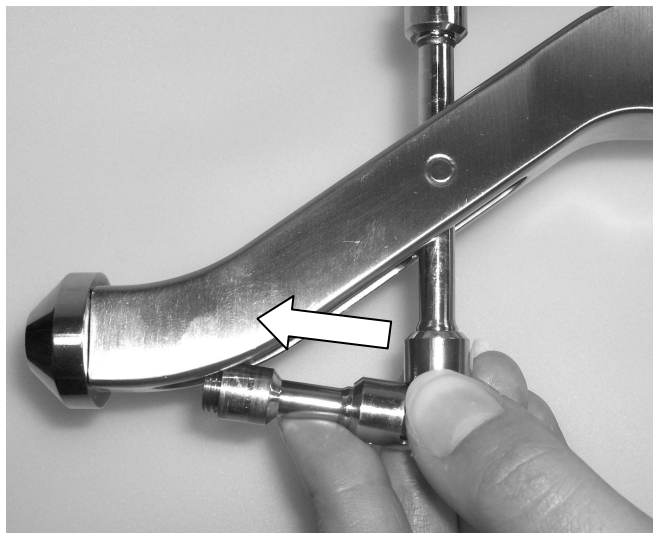

② Insertion of the drive chain

Position the drive chain into the body of the impactor so that the two pins engage properly into the slot.

